Two level Integrator Indicator

Quality Certification

Product code _____

LOT _____

Integration conditions

Temperature _____

Gas concentration _____

RH _____

SV (54 °C) - SAFE condition _____

SV (37 °C) - SAFE condition _____

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Product description

Integron® IT12 Integrator Indicator is a two level indicator, designed for Ethylene Oxide (EO) sterilization processes. The integration level contains a purple/brown ink which was developed to turn to green when a theoretical spore population reaches its kill time, thus indicating that the SAFE condition has been met.

Level 1 - Red: Exposure level, turns yellow to indicate exposition to the process, but does not ensure that the variables required to achieve the integration condition have been achieved (UNSAFE condition).

Level 2 - Purple / brown: Integration level, turns green to indicate that the integration condition indicated in the quality certificate has been achieved. This condition consists of the kill time of a theoretical 10⁶ *Bacillus atrophaeus* spore population, calculated in a BIER (Biological Indicator Evaluator Resistometer), and is called SAFE condition.

Characteristics

Consists of a paper strip of 60 mm x 25mm laminated on the back side.

Indications for use

Use the integrator indicator to monitor sterilization EO cycles. The SAFE condition in EO sterilization processes is achieved as described in quality certification.

Precautions

Do not use Integron® IT12 Integrator Indicator to monitor Dry Heat, Steam or sterilization methods other than EO. Do not reuse the sterilizer until the integrator color reaches SAFE condition.

Instructions for use

• Include Integron® IT12 Integrator Indicator along with the material to be sterilized in a suitable package according to the recommended sterilization techniques.

• Place Integron® IT12 Integrator Indicator in areas which are considered inaccessible to the sterilizing agent (EO).

• Sterilize in the usual way.

• Once the sterilization, evacuation and ventilation process is over, you may:

A. Remove the integrator indicator taking the necessary precaution to avoid contact with the remaining sterilizing agent.

B. Ventilate the load together with the integrator indicator and then do the reading.

• Analyze the results. If the SAFE integration condition has been reached, the sterilization process has been successful.

Storage and shelflife

• Indicators should be stored in dry places, away from light at temperatures between 10-30 °C, 30-80 % Relative Humidity. Do not wet. Do not store close to sterilizing agents.

• When stored as previously recommended, indicators expiration date is 5 years after the date of manufacture. Both the expiration date and the batch of the product are specified on the package label and in each unit of product. Do not use indicators after their expiration date.

• After use, the indicator color will remain stable for 1 year if stored at Temperature and Relative Humidity as previously indicated.